
INSTRUCTIONS

Wash the nozzle carefully in soap and warm water before each use. Rinse well and dry. Remove the screw-cap from the bottle of eye drops, remove the nozzle-cap from the eye-wash cup, screw the Eye Drop Dispenser onto the bottle, until it is tightly fixed. Use the cap supplied to cover the top of the bottle when not in use.

DO NOT TOUCH THE BOTTLE RIM WITH YOUR FINGERS.

- 1 Delicately position the Eye Drop Dispenser with the bottle above the eye, taking care that the upper part of the eye-dropper does not exceed more than a $\frac{1}{4}$ of the dispenser hole. Do not press the rubber dispenser against the face. The rim of the bottle should not touch the eye.
- 2 Keep the eye wide open. Tilt the head backwards to a horizontal position and look at the hole in the dispenser. Press out the prescribed number of drops. If tilting the head backwards to a horizontal position proves impossible, lie down.
- 3 Use the cap fixed to the Eye Drop Dispenser in place of the screw-on cap of the eye drop bottle.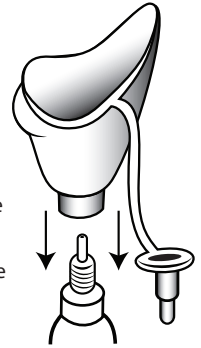

EYEWASH: The Eye Drop Dispenser may be used as a cup to wash the eyes. In this case, insert the yellow nozzle-cap (supplied) at the base of the Eye Drop Dispenser and use it as for any other eye wash cup. The white tip on the bottle of eye drops makes it possible for you to visibly determine that the cap has been entirely removed before use.

WARNING: If the Eye Drop Dispenser is used at any time with a different bottle, it is essential to wash the Dispenser again with soap and warm water and rinse it thoroughly to disinfect it and prevent any risk of contamination.

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