

Snore Solutions™

Set includes:

- Snore Solutions mouth guard
- Storage Case

Caution:

- Do NOT eat, chew or swallow mouth guard
- Keep out of reach of children

Note: Discontinue use of this product in the event of irritation. It is recommended to consult a doctor before continued use of this product

Directions:

1. Take the mouth guard out of the case and wash thoroughly using warm water for 2 minutes. Dry before using.
2. Gently place mouth guard in the mouth and adjust to comfortable position:
 - a. Do not bite
 - b. Do not cover the mouth guard with lips
Note: lips will naturally remain apart during use
 - c. Do not put tongue against the holes on mouth guard and ensure the six holes remain unblocked
3. Thoroughly wash and dry mouth guard with warm water after use. Place mouth guard back in storage case to protect from elements until next use.

General Care:

- Store with care to prevent damage
- Clean with damp cloth
- Wash mouth guard in warm water

Note: There are no serviceable parts in this product

PLEASE KEEP THESE INSTRUCTIONS IN A SAFE PLACE FOR FUTURE REFERENCE

Designed in USA

Made in China

Production Run: Please refer to English instruction for production run number